

USER MANUAL

Product Name: Disposable Respirator Mask (None - Medical)

Material: Nonwovens, meltblown fabrics and hot air cotton

Specification:

1. Dust-free Filtration layer Protections
2. Electrostatic filter Enhanced filtration of Micro Particles
3. Activate Carbon Layer Absorb Smaller Pollutants
4. 2nd Electrostatic filter Double Reinforcement to Prevent Additional Particles
5. PP Non-woven Fabric Layer for Soft Skin-friendly Comfort. (NOT FOR MEDICAL USE)

WEARING METHOD:



- *Face the side of mask without nose clip jet the nose clip up.
- *Hold the mask with hands and fix it against the chin.
- *Put your fingers on the middle of the metal nose clip, while pressing inward, move the finger to both, till metal nose closely touch the nose bridge.
- *Pull the earband and hang on both ears.
- *Check the tightness of the mask on the face.

Validity Period: 3 Years

Storage Environment: Unopened products should be stored at a temperature of -20°C to 30°C, and relative humidity below 80%

CAUTION!

This product is only used for respiratory protection of some particles, misuse can lead to disease or even death. Please use in proper method.

Patients with dyspnea or congenital heart should use it carefully. This product cannot be use as gas mask in toxic enviornment.

Non-Medical Type. for personal protection only.

Made in China